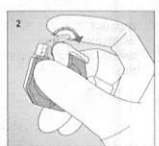
ES Instrucciones de uso de la lanceta de seguridad para incisión en el talón	PT Instruções de utilização da lanceta de segurança para punção do calcanhar	ET Toote Heel Incision Safety Lancet kasutusjuhend	LV Lietošanas instrukcija: lancete ar drošības mehānismu iegriezumiem papēdī	LT Saugaus lanceto kulno incizijai naudojimo instrukcija	EL Οδηγίες χρήσης για Νυστέρι ασφαλείας εκτομής φτέρνας	HEB הוראות שימוש למחט בטיחות לחיתוך בעקב	AR تعليمات الاستخدام لمشرط سلامة شق الكعب
---	---	---	---	---	--	---	--

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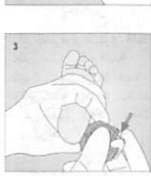
Select a site identified in the colored area. Clean the incision site of the heel with an antiseptic swab and allow skin to air dry.


Release trigger lock mechanism by pushing down the safety tab.

Raise the foot above the baby's heart level. Place the blade slot surface of the device flush against heel site. Ensure that both ends of the device in contact with the skin, and depress the trigger.

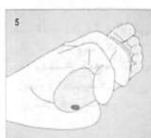After triggering, immediately remove the device from infant's heel and discard it into a biohazard sharps container.

First drop of blood should be wiped away gently with a dry sterile gauze pad. Apply gentle pressure with the thumb and ease intermittently as drops of blood form.

Following blood collection, gently press a dry sterile gauze pad to the incision site until bleeding has ceased.

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Warning:
1. Do not repeat blood sampling at the same incision site.
2. Do not use in infants with edema, infection or inflammation at the incision site.
3. Do not use in infants with skin diseases or skin lesions at the incision site.
Contraindications:
Unknown.

Cautions:
1. Do not use if the individual packaging has been previously damaged.
2. Check the use-by date on the packaging, and do not use the lancet beyond the use-by date.
3. The safety lancet is for disposable use and do not reuse.
4. Discard the used lancet into a suitable sharps container.

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